


REF : PRE100-S

PREVASSE

Pressure Redistributing Cushion

Prevasse is an aid in the prevention of pressure ulceration for all patients including those assessed as high risk, and for the treatment of existing stage 1 and 2 pressure ulcers. Clinical supervision is advised if damage is severe. Prevasse may be cleaned and reused for multiple patient use.

1. Insert the pump nozzle firmly into the cushion valve
2. Inflate with **just** enough air to ensure the patient is 'floating' on the air cushion, by adding a small amount of air at a time. **DO NOT OVER INFLATE.**
3. Close the valve when finished and invert valve into cushion so as to be out of the way
4. Position cushion with strap at rear and arrow pointing forwards. Secure around the back of the chair.

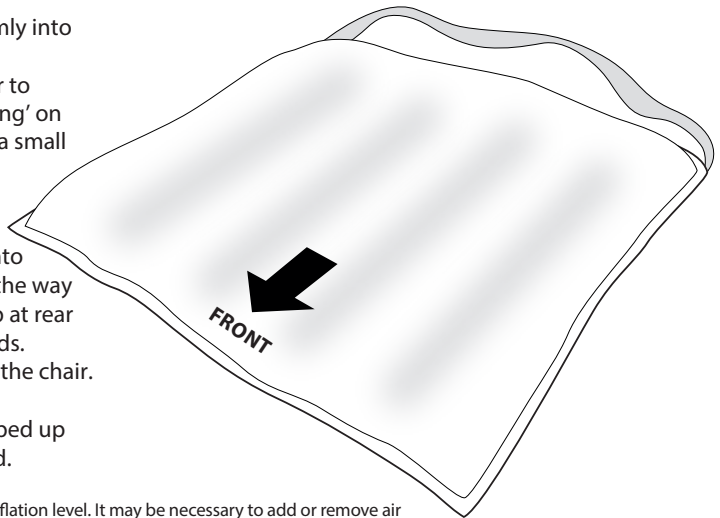

If required, Prevasse may be topped up by hand with the pump provided.

Check device regularly to ensure proper inflation level. It may be necessary to add or remove air for just the right comfort level, but always ensure the patient is fully supported on the cushion.

⚠ If deterioration in skin condition is noted, cease use and seek clinical advice.

In the unlikely event that you discover a product fault, please contact ProSys International Ltd or the UK Competent Authority (MHRA).



Precautions

Keep away from sharp objects, pets, babies and small children, and avoid sources of heat (including lighted cigarettes). Product does not contain latex.

Please note, Prevasse is not suitable for the following:

- Persons weighing in excess of 139 kg
- Persons with unstable spinal fracture
- Where body shape is such that the person cannot be fully supported by the cushion





Disposal: Please check with your National or Local Authorities regulations prior to disposal.



ProSys International Ltd. Suite 303, Highland House, 165 The Broadway, Wimbledon, London, SW19 1NE, UK • Tel: +44 208 944 7585

Cleaning instructions

Cushion may be cleaned using lukewarm water and soap or detergent only. Disinfect in accordance with local policy for standard mattresses but do not use phenol or alcohol based disinfectants as these may compromise the device. Always rinse well with fresh water after cleaning, and always allow to air-dry at room temperature prior to reuse or storage.

Do not dry clean  tumble dry  machine wash 
or iron 

Materials: Thermoplastic Polyurethane

